



The water for grinding in tank must be replaced with clean water after approximately 100 lenses are ground.

- 1- Open the door of the table and take the tank with pump out.
- 2- Disconnect the water supply pipe, drain pipe and power cord for pump.
- 3- Water supply pipe is disconnected by pressing the red button.
- 4- Remove the water supply pump from the tank and lay it on its side.
- 5- Remove the cover and drain the skimmed water only.
- 6- Dispose of sediment ground wastes as non-combustible.
- 7- Fill the tank with water two inches from the top of the tank
- 8- Replace the cover and the water supply pump to their places.
- 9- Place the water supply pump and tank into the table.
- 10- Connect the water supply pipe, drain pipe and power cord for pump as before.